

BIOCLEAR

Dual Color Disclosing Solution

For Biofilm Identification, Biofilm Removal, and Crack Identification

Instructions for Use:

1. Thoroughly dry teeth.
2. Using a brush applicator, apply disclosing solution to tooth surfaces.
3. Rinse. Old biofilm will stain Purple, newer biofilm will stain Pink.
4. Effectively remove biofilm with the Bioclear Blaster.

Storage Conditions: Do not store below 32°F/0°C or above 212°F/100°C



BIOCLEAR MATRIX SYSTEMS, LLC
3901 S FIFE ST, STE 200
TACOMA, WA 98409
USA



Made in the USA



[www.bioclearmatrix.com/
resources/ifu-sds/](http://www.bioclearmatrix.com/resources/ifu-sds/)



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



ID: 70322